





SAFE-ASSIST® SHOWER SEAT

SAFE-ASSIST® Shower Seat Models

When safety in the shower is paramount, Con-Serv SAFE-ASSIST® shower seats are the industry recognised solution with non-slip foldable seat, rounded edges and water drain channels.

Manufactured from grade 304 Stainless Steel, Polar White UV resistant High Density Polyethylene (HDPE) seat top and assembled with Tension Grip Hinge, ensuring stability up to 160 kg. Special sizes available on request.

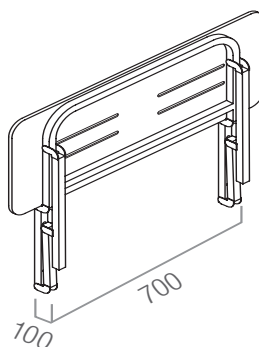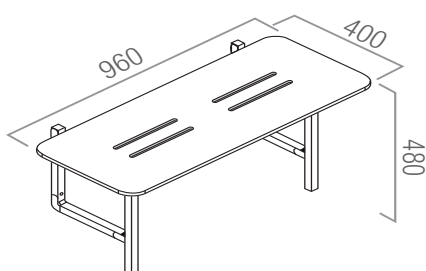

SS 964

960 mm x 400 mm

Max Load: 160 kg (25 Stone)


White Seat/Stainless Steel Frame

Compliant to Australian Standard AS 1428.1



Safe Assist® Folding Shower Seats

Model: SS 964SS Satin Stainless 960 x 400mm Folding Shower Seats

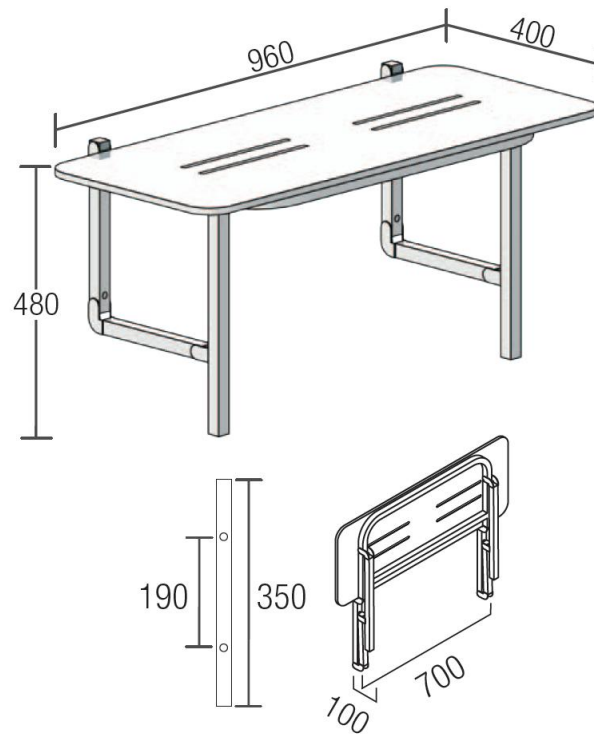

Features:

- 960 x 400mm Polar White High Density Polyethylene (HDPE) Seat with self draining channels
- Frame fabricated from 304 Stainless Steel
- Complies with requirements of AS 1428.1-2009 for a two walled shower enclosure
- Stainless steel fixings supplied
- MAX. LOAD 160 kg (25 stone)


Please note: Seat deck should be mounted 470mm to 480mm ABOVE THE FLOOR

Installation Instructions

MUST BE FITTED TO A SOLID WALL OR OTHER FIRM SURFACE.

For fixing to timber, use only hexagon coach screws. For fixing to brickwork, block work or concrete, use the hexagon coach screws with 10 mm wall plugs or equivalent.

Grade 304 Stainless Steel M8 x 75 mm hexagon coach screws and 10 mm wall plugs supplied.

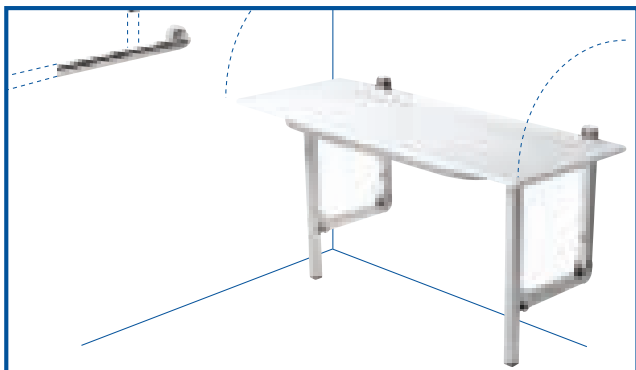

Accessible - Install seat as per AS 1428.1 Design for Access and Mobility. When in the horizontal position ensure seat does not present a hazard and the grab rail is accessible.



Fixing - Screw wall mount to wall with stainless steel coach screws. Note: On stud wall, make sure wall mount is fixed into studwork and noggins. For masonry walls, use wall plugs supplied.



Positioning - In the horizontal position, level check and spot mark wall through seat frame mount holes.



Checking - Check seat is firmly secured to wall and the Tension Grip Hinge screws are appropriate for ease of movement.

Warranty

Con-Serv Corporation Australia Pty Ltd warrants all products to be free from manufacturing defects for 5 years from date of purchase. Con-Serv will repair or replace defective items if defects occur in normal use. Con-Serv will not be liable for defects arising through normal wear and tear. Item(s) must be returned with proof of purchase to Con-Serv at 17 Glentanna Street, Kedron, Queensland 4031. We will attend to your warranty claim after we make appropriate investigations. Contact our Service Manager during normal office hours on 1300 467 322 or at our email address of warranty@con-serv.com.au. If a warranty claim is accepted Con-Serv will bear the reasonable costs of the return of the defective item(s) to us. A claim for expenses associated with making a claim for a defective product must be in writing and submitted to us with copies of supporting documents. The benefits given under our warranty are in addition to other rights and remedies of the customer under a law in relation to goods and services to which this warranty relates. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or a refund for a major failure and compensation for any other reasonable foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.



CON-SERV CORPORATION AUSTRALIA PTY. LTD.

17 Glentanna Street, Kedron Brisbane, Qld 4031 Australia

Phone: +61 (0)7 3630 5744 • Fax: +61 (0)7 3630 5755

Email: mail@con-serv.com.au • Web: www.con-serv.com.au