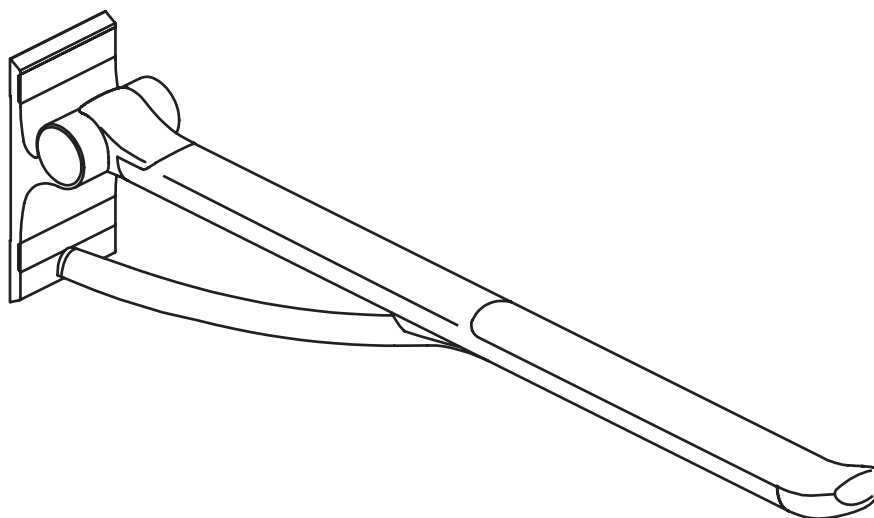
(EN) Mounting instruction

(DK) Monteringsvejledning

(DE) Montageanleitung

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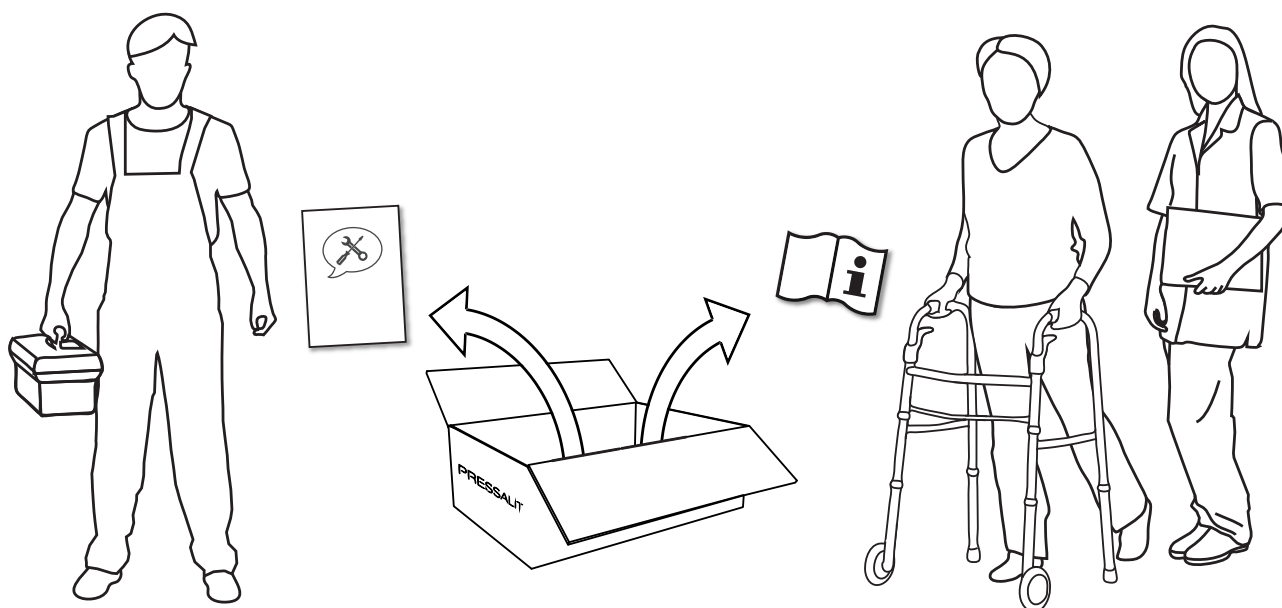
(NL) Montage-voorschrift

(SV) Monteringsanvisning

(NO) Monteringsveiledning

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Subject to alteration without further notice. Ret til ændringer forbeholdes. · Recht auf Änderungen sind vorbehalten. · Nous nous réservons le droit d'apporter toutes modifications sans préavis. · Wijzigingen voorbehouden. · Rätten till ändringar förbehålles. · Vi forbeholder oss rett til endringer. · Salvo variaciones.

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EN Mounting Instruction

Please read and carefully follow these instructions. Find the product number on the product label and ensure that these assembly instructions match your product.

WARNING: Failure to comply with these instructions may result in accidents involving serious personal injury or the product being damaged.

WARNING: If you suspect that any individual parts may be damaged, contact your Pressalit dealer or service provider. Do not remove, attempt to repair or install parts yourself, unless directed to do so by Pressalit or its agents.

WARNING: No modification of this equipment is allowed without express authorization from Pressalit.

WARNING: These instructions must be followed, and the product must be installed professionally by a qualified person; e.g. a licensed plumber, in order to avoid personal injury or property damage while using the product.

WARNING: Pressalit cannot be held liable for injury to persons or damage to property caused by faulty mounting.

Installation location

This product is designed for indoor use in a wet environment, such as a bathroom or washroom. When choosing an installation location, ensure that there will be adequate space for access to the product.

Assembly and installation

Mount the product against a wall in accordance with the illustrations in this assembly instruction. Contact your Pressalit agent, if assistance is needed when installing the product.

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




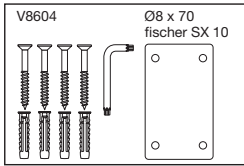
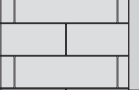
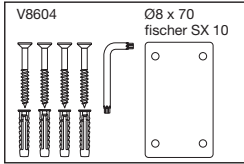
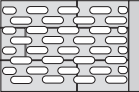
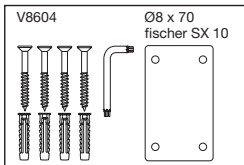
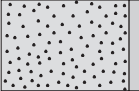
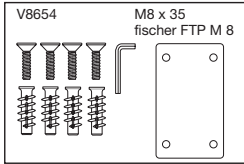
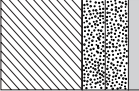
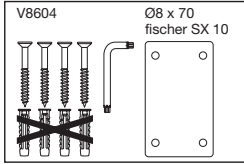
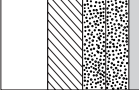
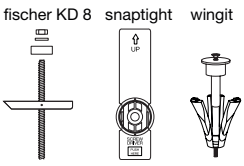
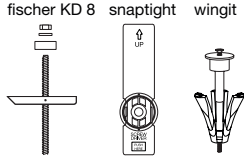
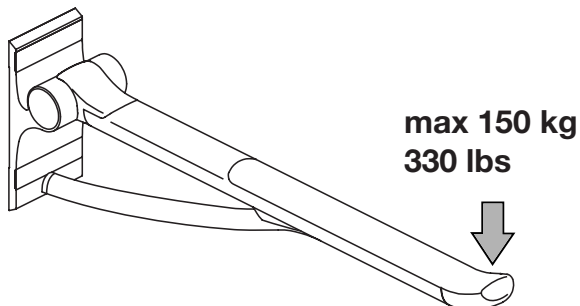
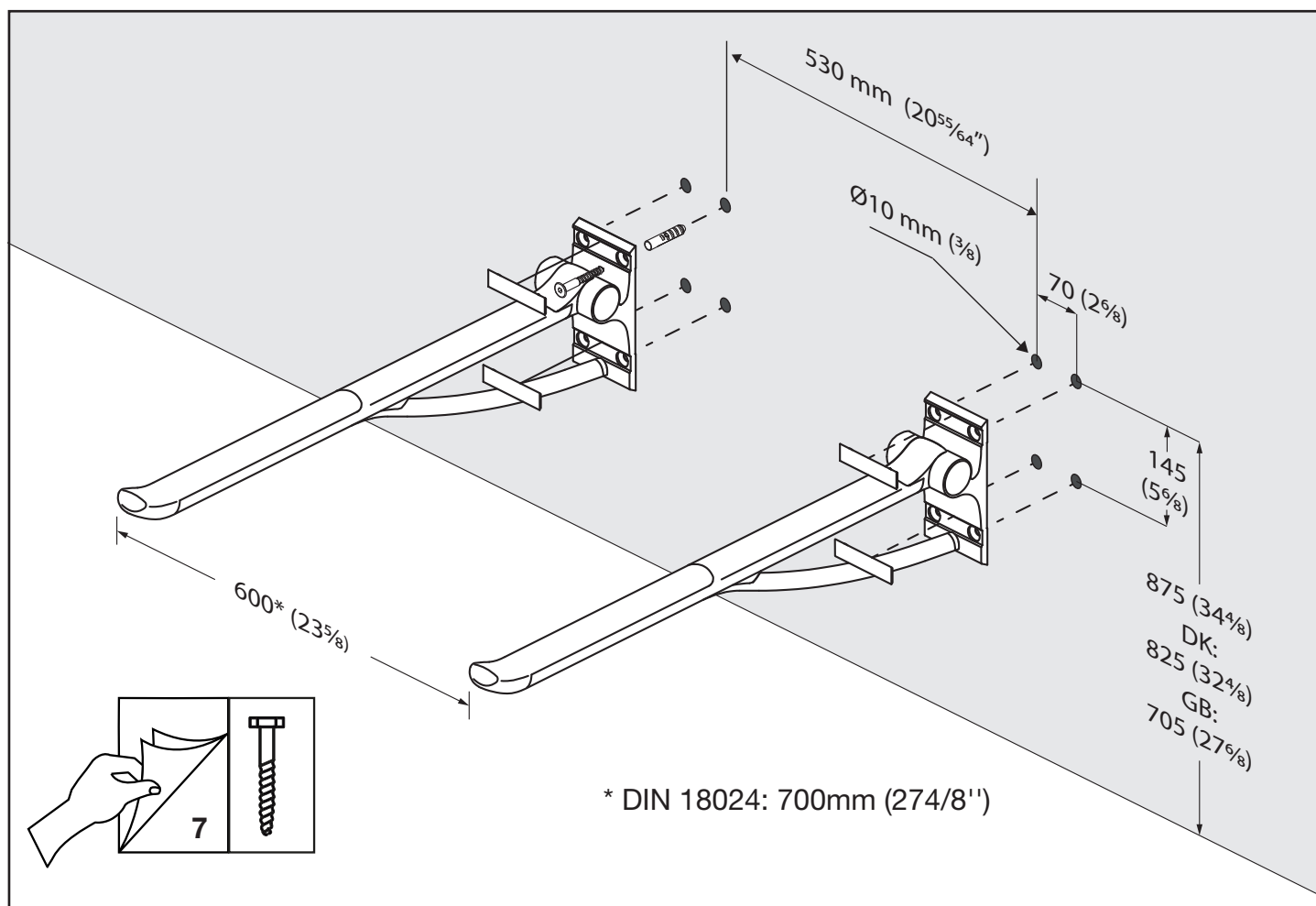
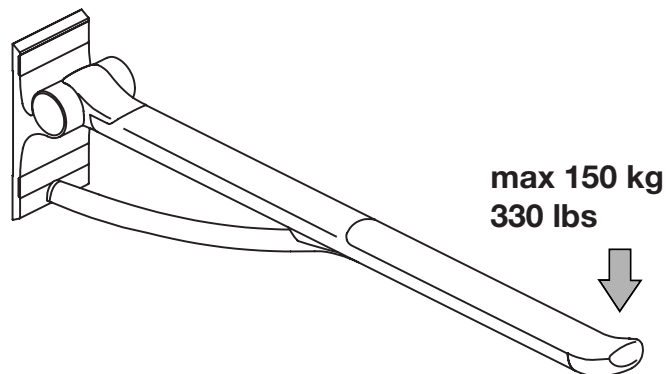
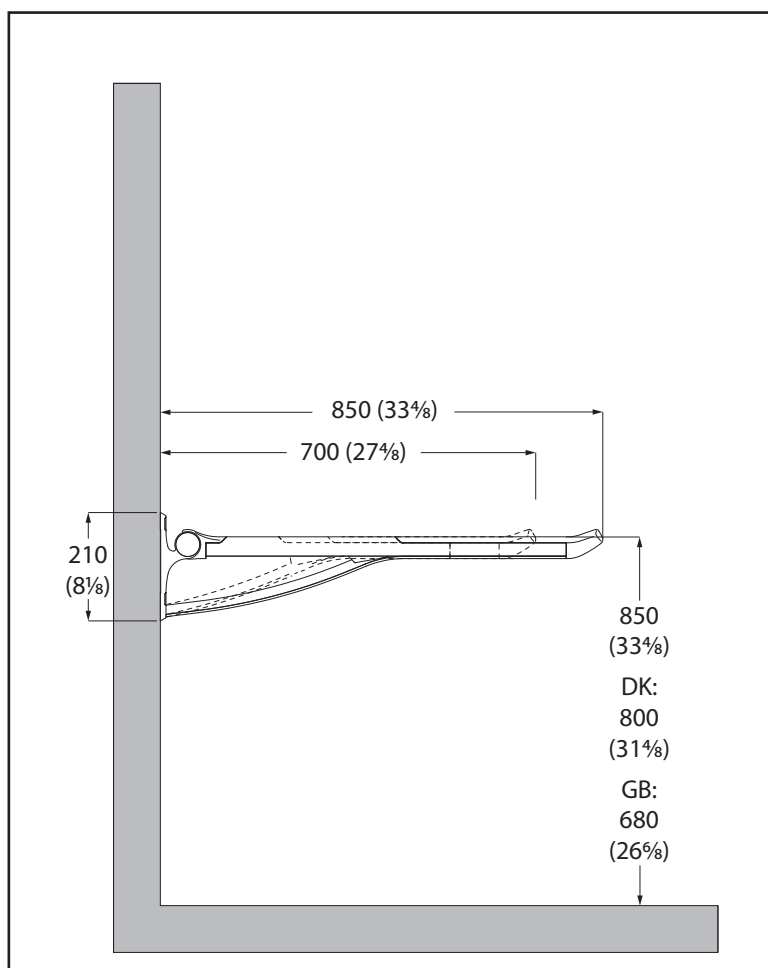
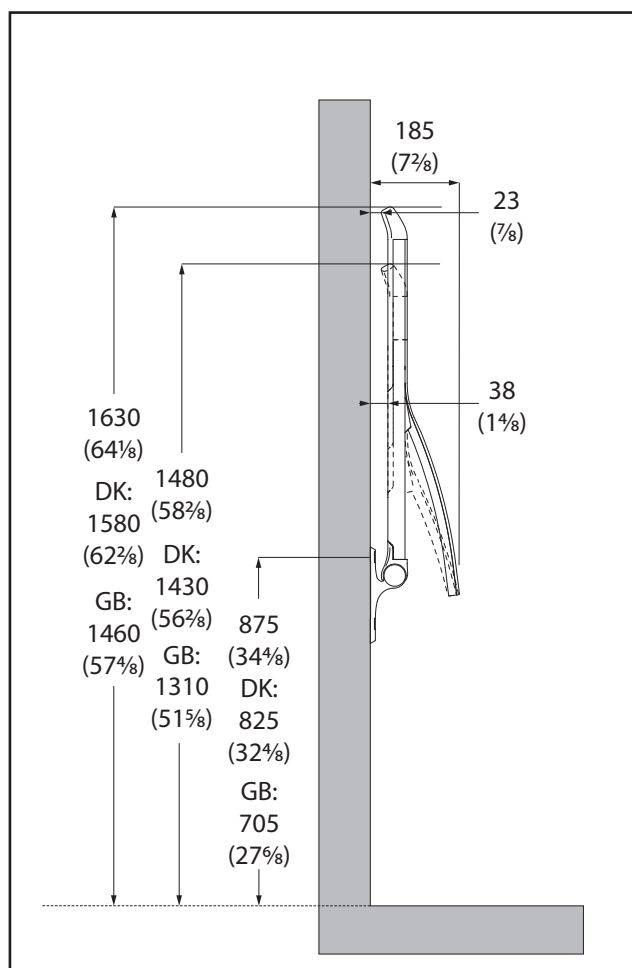
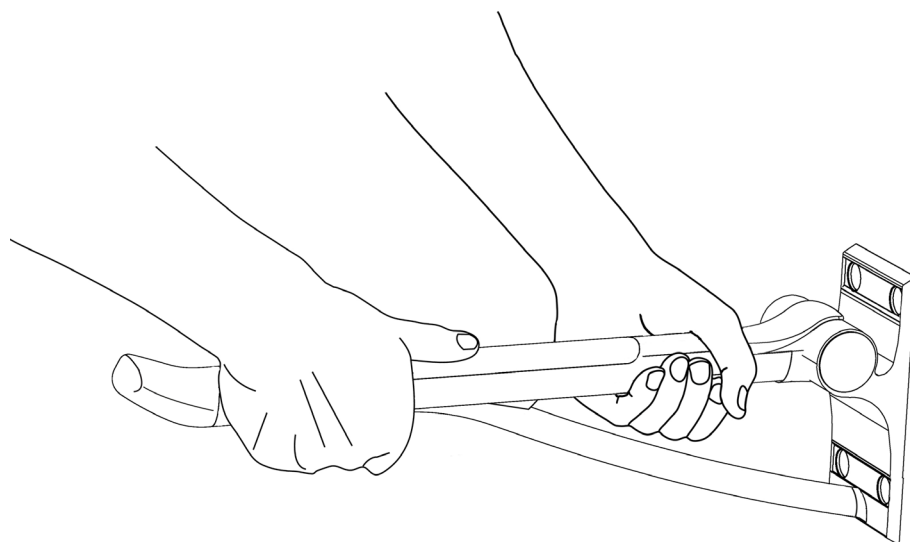
R360070

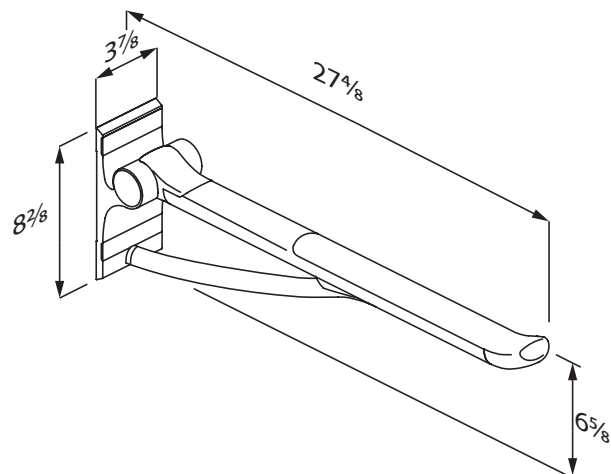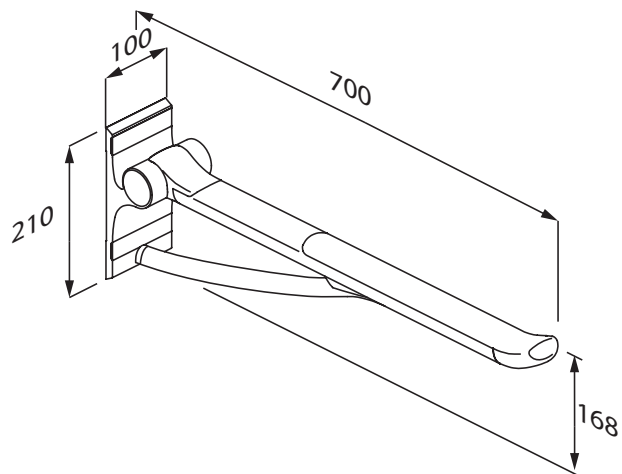R360085

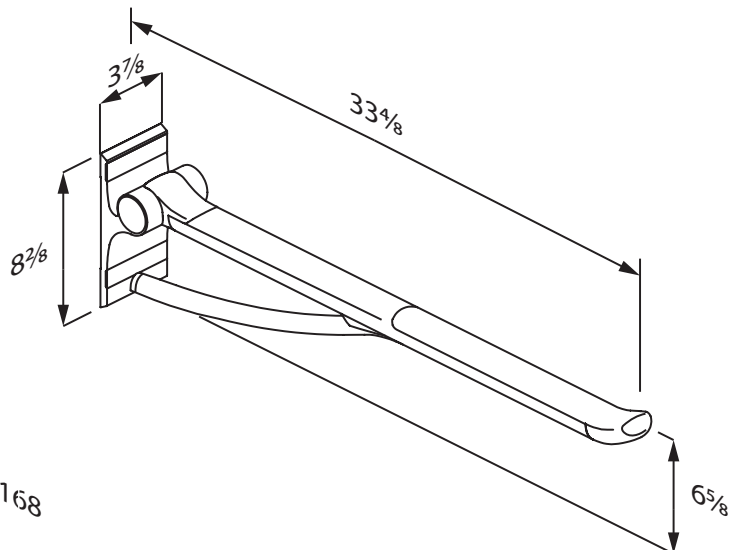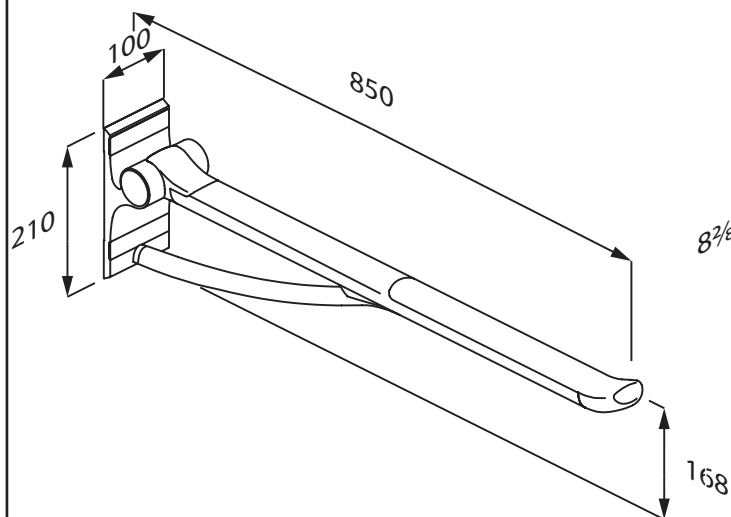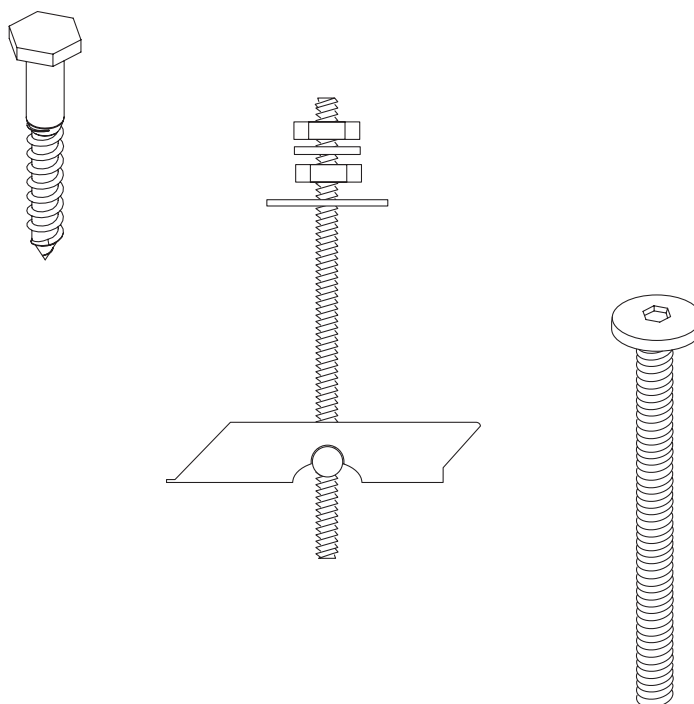(EN) Load bearing installation recommendations

- Pressalit can only be held responsible for installations carried out by Pressalit's own installers.
- Pressalit can not be held responsible for reinforcements made in a faulty manner.
- The below recommendations are based on experience and internal test results.
- The below recommendations are to be considered as guidelines only.
- Subject to alteration without further notice.

This advisory document shows the maximum load capacity for individual bolts and other types of anchors that can be used for fixing Pressalit's products to the wall.

The maximum load capacities shown below are per bolt/anchor and must be considered in relation to the information provided for the product that is being fitted. The information specific to the product will show how great the load is on each fixing point when the product is used at its recommended safe working load.

Only if the load on each fixing point of a given product is less than the maximum load capacity of the type of bolt/anchor under consideration, should the bolt/anchor in question be used.

E.g.

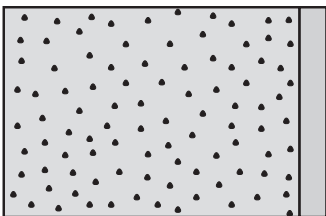
One might wish to fit a nursing bench 3000 to an aerated concrete wall:

Maximum load

Maximum load: 200 kg.

When the bench is loaded with 200 kg, each mounting bolt supports 26 kg.

Each fixing point is here carrying a load of 26 kg, and the bolts typically recommended for aerated concrete (Ø10, M8, M10) all have a maximum load capacity that is higher than 26 kg, so they can all be used.









Aerated Concrete P4	Plug	Anchor / Bolt	Maximum load per anchor
	fischer GB 14 	Ø10 x 80-105 	90 kg.
	fischer FTP M 8 	M8 x 35 	65 kg.
	fischer FTP M 8 	M8 X 35 	65 kg.
	fischer FTP K 6 	M5 x 50 	40 kg.









Please see the mounting instructions and the technical datasheets (both can be downloaded from our website) for information about which types of bolts/anchors are recommended for use on which type of wall.

Fixing kits to be specified and ordered separately.

They do not come included with the product.

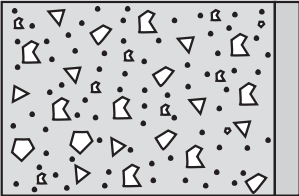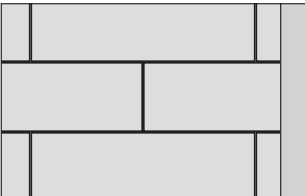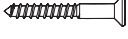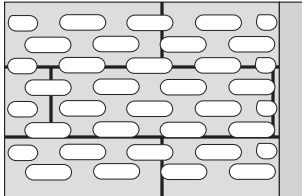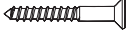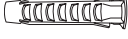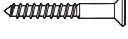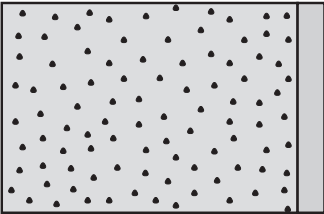
Choose the appropriate anchor for your wall

Concrete > B25	Plug	Anchor	Maximum load per anchor
	fischer SX 12 	Ø10 x 70/80 	170 kg.
	fischer SX 10 	Ø8 x 70/80 	120 kg.
	fischer SX 6 	Ø5 x 50 	65 kg.
Solid brick >= Mz12	Plug	Anchor	Maximum load per anchor
	fischer SX 12 	Ø10 x 70/80 	70 kg.
	fischer SX 10 	Ø8 x 70/80 	60 kg.
	fischer SX 6 	Ø5 x 50 	30 kg.
Solid sand lime brick >= Hlz12	Plug	Anchor	Maximum load per anchor
	fischer SX 12 	Ø10 x 70/80 	170 kg.
	fischer SX 10 	Ø8 x 70/80 	120 kg.
	fischer SX 6 	Ø5 x 50 	50 kg.
Aerated Concrete P4	Plug	Anchor / Bolt	Maximum load per anchor
	fischer GB 14 	Ø10 x 80-105 	90 kg.
	fischer FTP M 8 	M8 x 35 	65 kg.
	fischer FTP M 8 	M8 X 35 	65 kg.
	fischer FTP K 6 	M5 x 50 	40 kg.

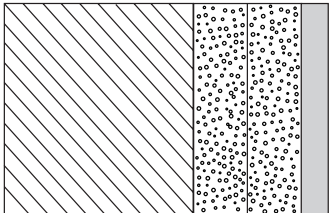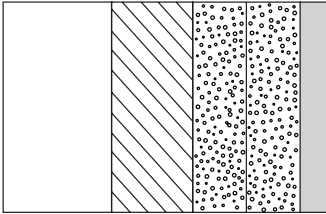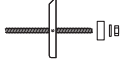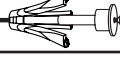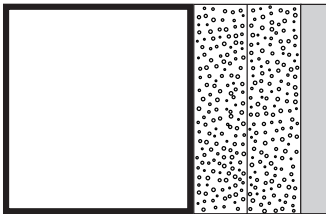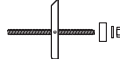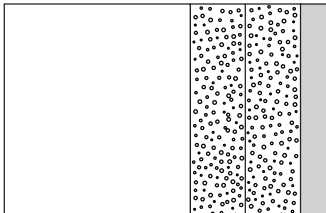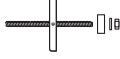
Drywall (2 X 12,5 mm) water-resistant, re-enforced with wood	Anchor	Maximum load per anchor
	Ø10 x 70/80 	120 kg.
	Ø8 x 70/80 	100 kg.
	Ø5 x 50 	40 kg.
Drywall (2 X 12,5 mm) water-resistant, re-enforced with 22 mm plywood	Anchor	Maximum load per anchor
	fischer KD8 toggle 	133 kg.
	Wingit 	133 kg.
	Snaptight 	133 kg.
Drywall (2 X 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)	Anchor	Maximum load per anchor
	fischer KD8 toggle 	94 kg.
	Wingit 	94 kg.
	Snaptight 	94 kg.
Drywall (2 X 12,5 mm) water-resistant	Anchor	Maximum load per anchor
	fischer KD8 toggle 	85 kg.
	Wingit 	85 kg.
	Snaptight 	85 kg.

Reinforcement of wall with 40-50 mm wood

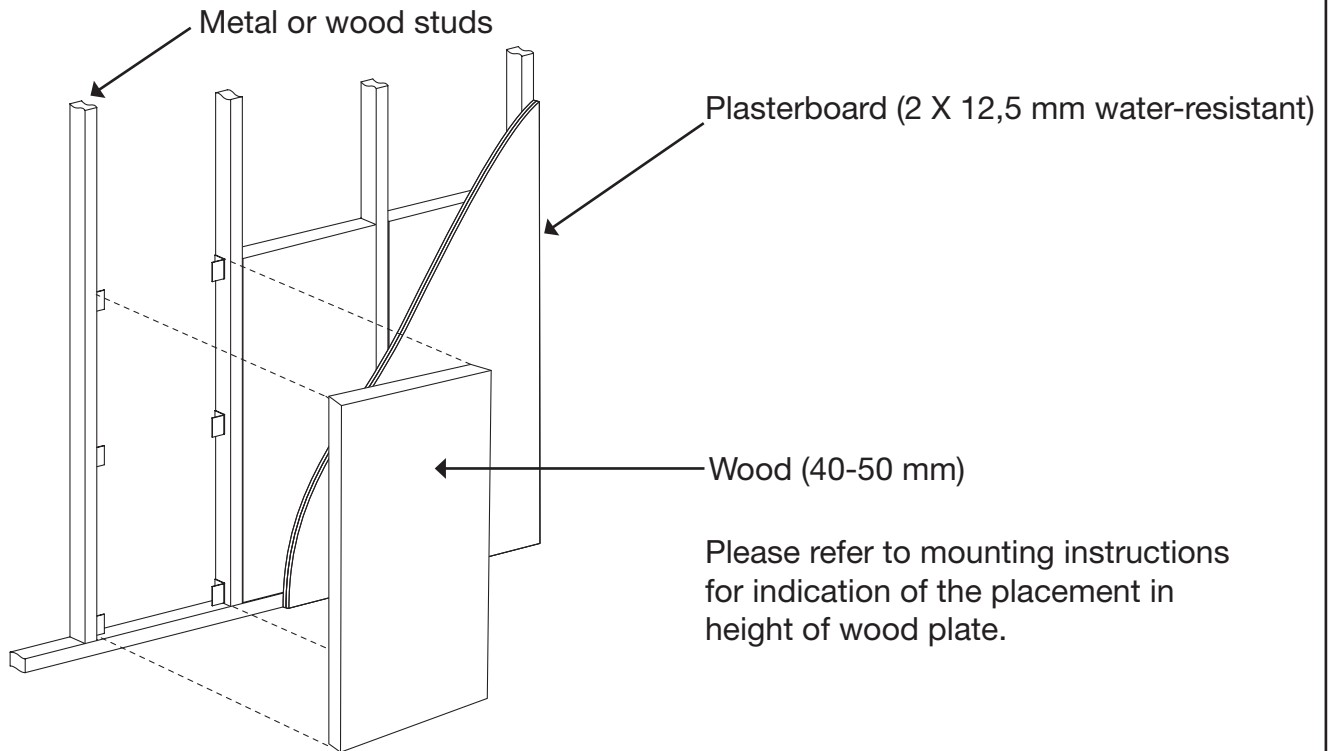Reinforcements of walls should always comply with national standards.


Wall cross-section

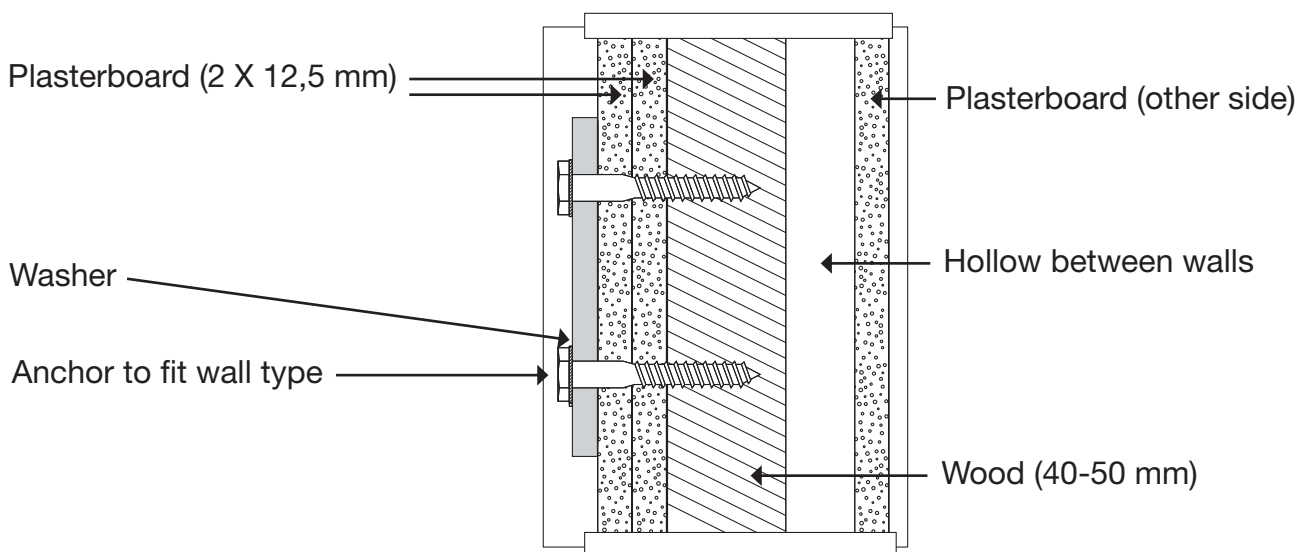
Reinforcement of wall with plywood

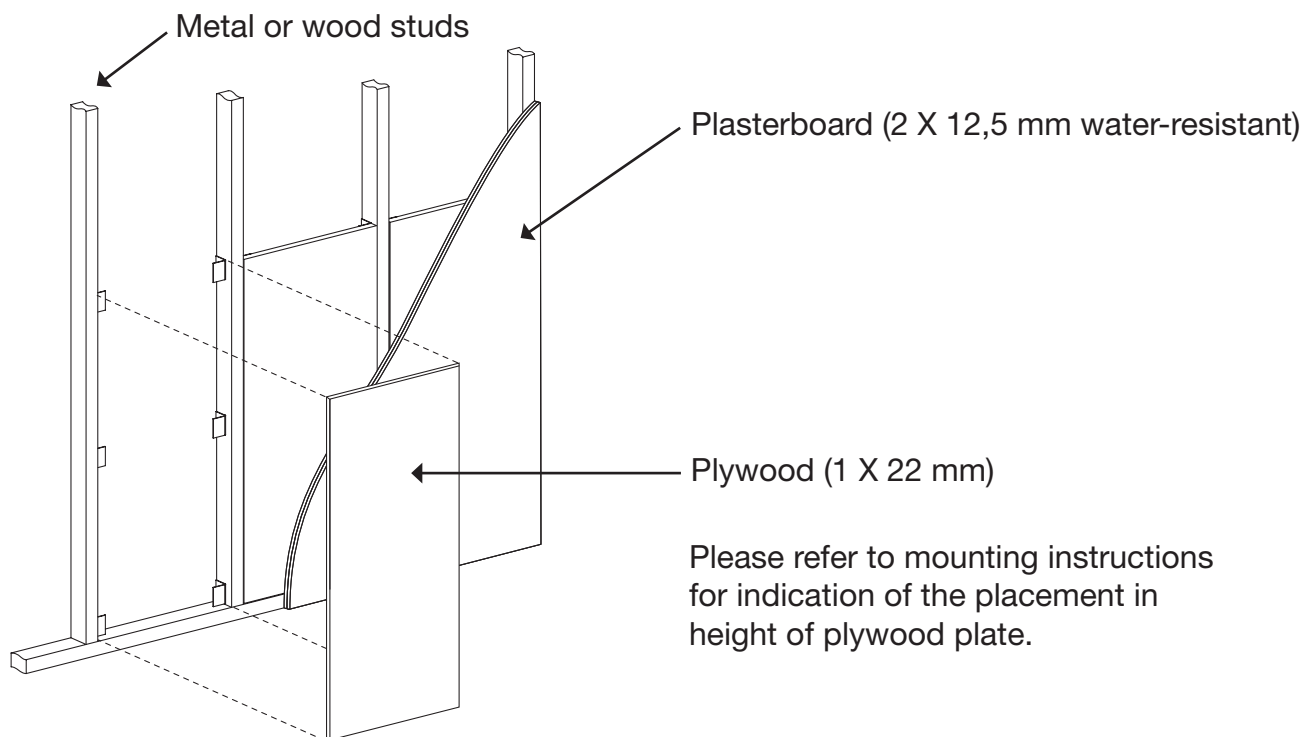Reinforcements of walls should always comply with national standards.




Wall cross-section

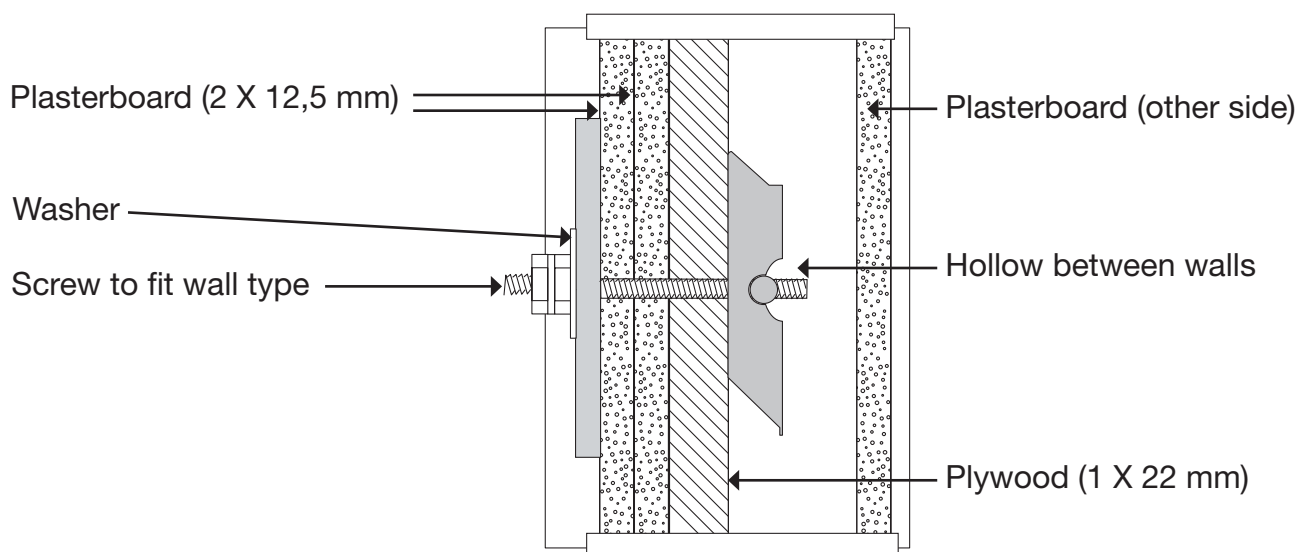Reinforcement of wall with steel

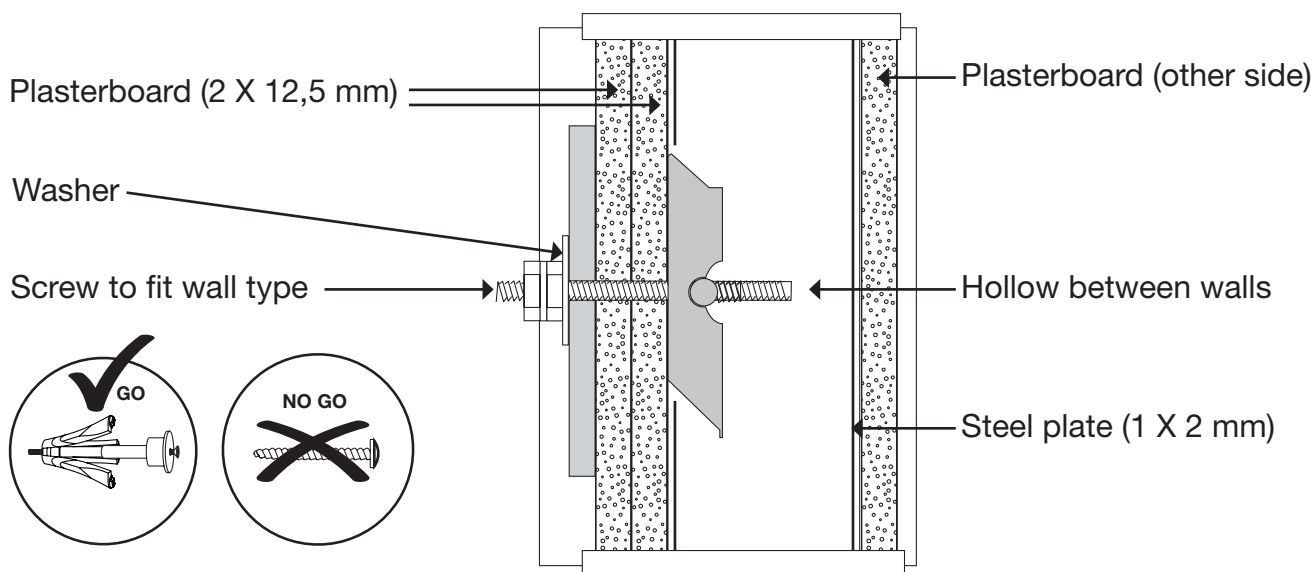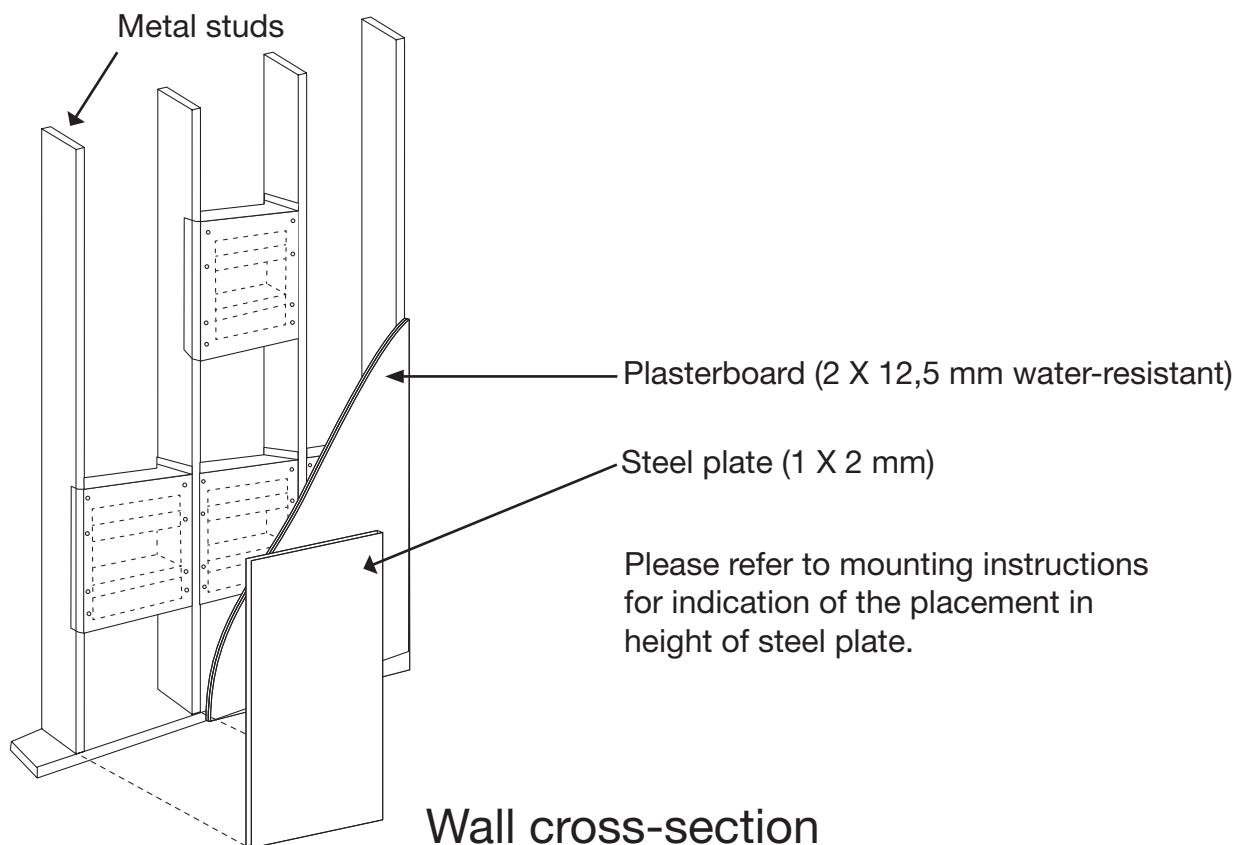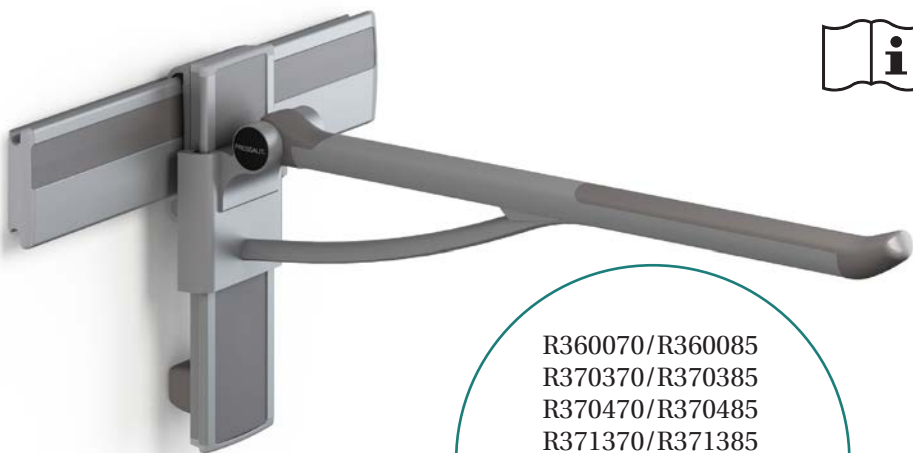Reinforcements of walls should always comply with national standards.

R360070/R360085
R370370/R370385
R370470/R370485
R371370/R371385
R371470/R371485
R381470/R381485
R391370/R391385
R391470/R391485

PLUS support arm – Operation and maintenance manual	en
PLUS toiletstøtte – Brugs- og vedligeholdelsesvejledning	da
PLUS Stützklappgriff – Gebrauchs- und Pflegeanleitung	de
PLUS barre de relèvement – Manuel d'utilisation et d'entretien	fr
PLUS toiletsteun – Gebruiks- en onderhoudshandleiding	nl
PLUS toalettarmstöd – Bruks- och underhållsanvisning	sv
PLUS toalettstøtte – Bruks- og vedlikeholdsveiledning	no
PLUS apoyabrazos - Instrucciones de uso y mantenimiento	es
PLUS podpora – Instrukcja obsługi i konserwacji	pl
PLUS 臂撑 - 操作和维护手册	中文

English	3
Dansk	22
Deutsch	39
Français	56
Nederlands	73
Svenska	90
Norsk	107
Español	124
Polski	141
中文	160

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Content

Mounting instruction	3
Operational signage	3
Symbols used in this manual	4
General safety	5
Product label	6
Versions	7
Intended use	9
Intended user profile	9
Operating environment	9
Operating instructions	10
All support arms	10
Height adjustable support arms	10
Sideways adjustable support arms	11
Height and sideways adjustable support arms	11
Dismounting from wall track	12
Mounting on wall track	12
Dismounting from vertical track	13
Mounting on vertical track	13
Cleaning	14
To remove limescale	14
Maintenance	15
Trouble-shooting	15
Technical data	15
Dimensions, mm	16
Dimensions, inches	18
Product weights	20
Disposal and recycling	21

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressalit.com.

Operational signage

An operational signage can be downloaded from www.pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning.

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

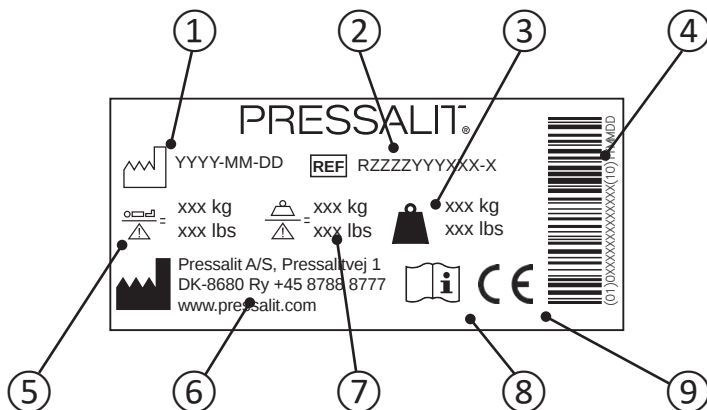
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit Care cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

① Production date

② Article number

③ Product weight

④ GS1-128 barcode

⑤ Maximum user weight

⑥ Name and address of manufacturer

⑦ Safe working load

⑧ Consult manual for use

⑨ This product is CE-marked in accordance with the European Medical Device Regulation 2017/745. A declaration of conformity can be found at www.pressalit.com.

Versions

Product label placement ①.

R360070

R360085

R371370

R371385

R371470

R371485

R370370

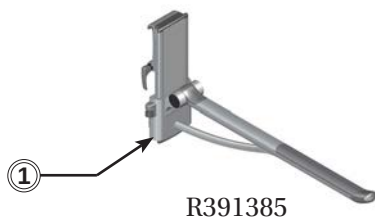
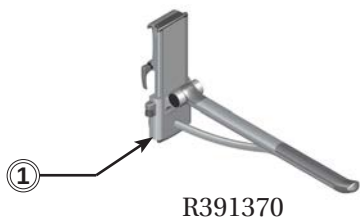
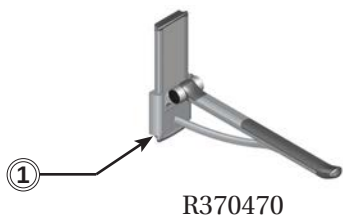
R370385

en

Versions

Product label placement ①.

Intended use

The support arm is used as support for a person when he or she sits down or gets up from a toilet.

Intended user profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room, in private homes, nursing homes and hospitals. For institutional use, the expected life time is 10 years.

Life cycle test performed according to ISO 17966:2016.

NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 5 years to 1 year.

Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.

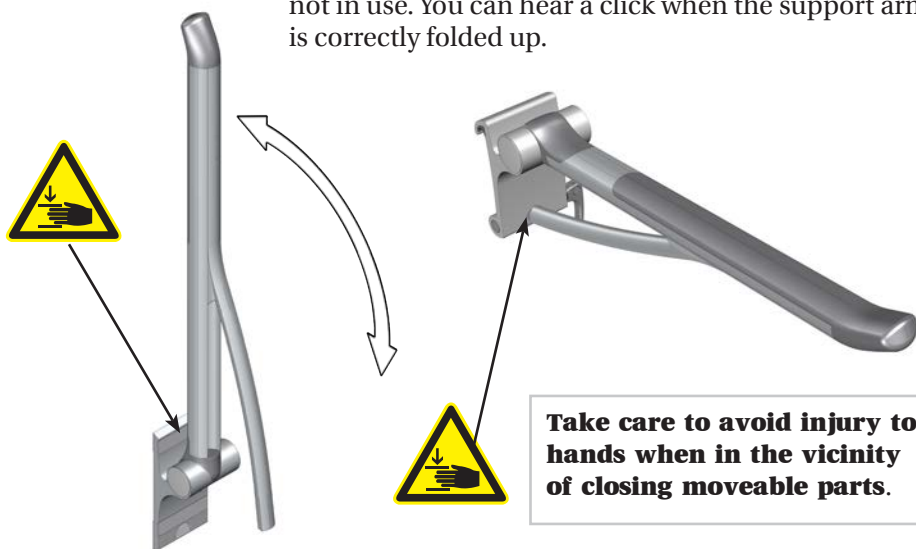
CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

Operating instructions




All support arms

Fold the support arm fully down. Fold it back up when not in use. You can hear a click when the support arm is correctly folded up.


Take care to avoid injury to hands when in the vicinity of closing moveable parts.

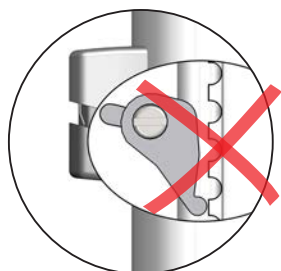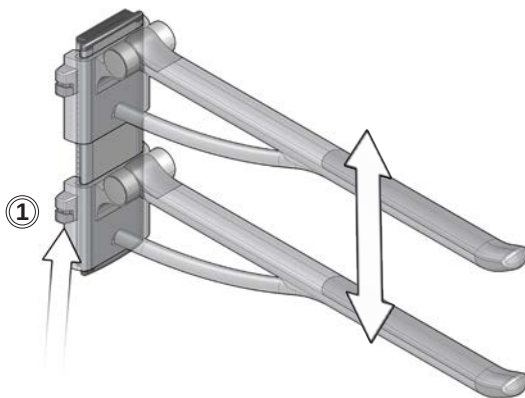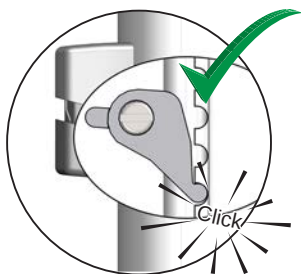en Height adjustable support arms

To adjust upwards, pull the product upwards.
To adjust downwards, pinch the locking button ① and hold it closed while moving support arm down. Release the locking button to secure the support arm in place. Make sure that the locking button pin is engaged in the notch.

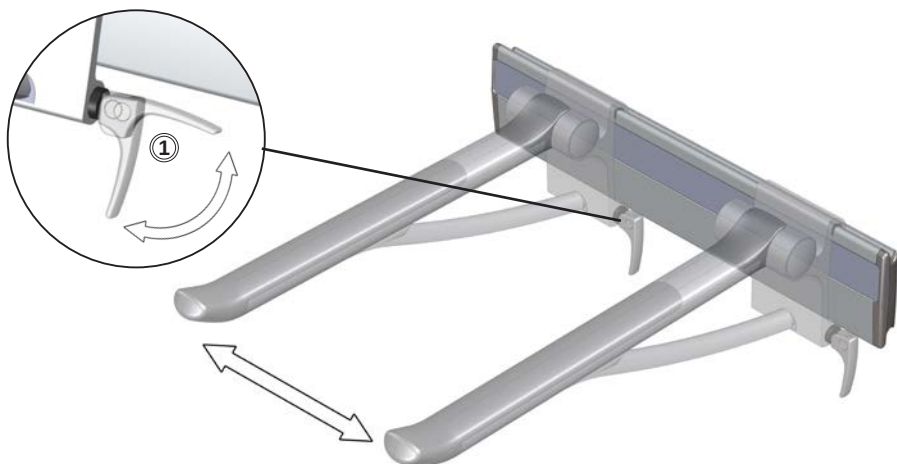
Sideways adjustable support arms

Release the locking handle ① and move the support arm sideways. Tighten the locking handle again to secure the support arm in place.

Height and sideways adjustable support arms

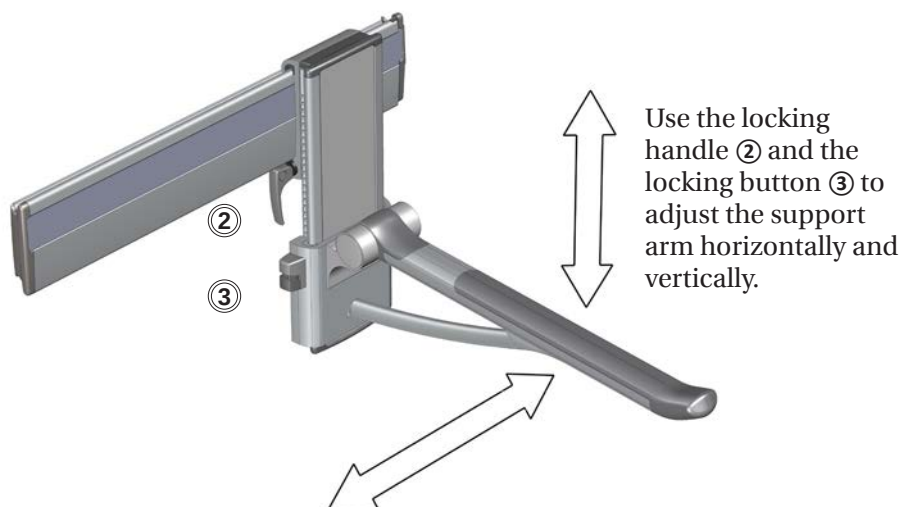
en

Use the locking handle ② and the locking button ③ to adjust the support arm horizontally and vertically.


Dismounting from wall track

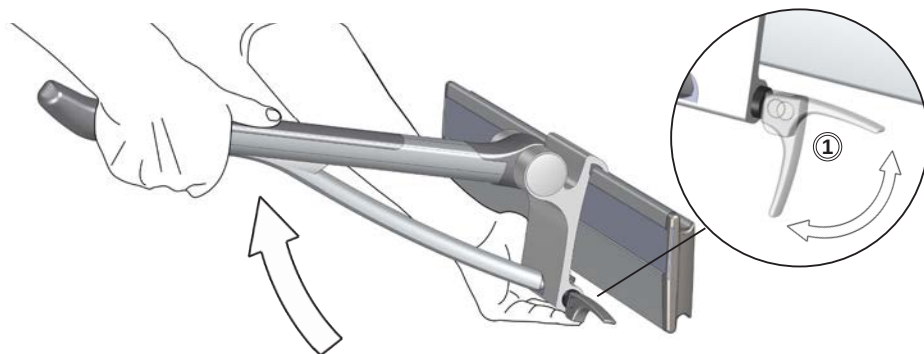To remove the support arm from the wall track, release the locking handle ① and lift the support arm upwards.


Mounting on wall track

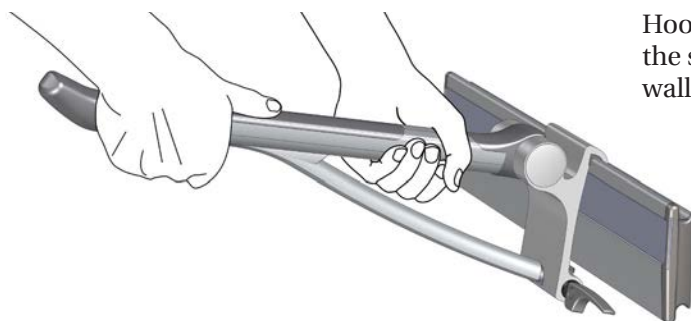
Hook the upper part of the support arm on the wall track.

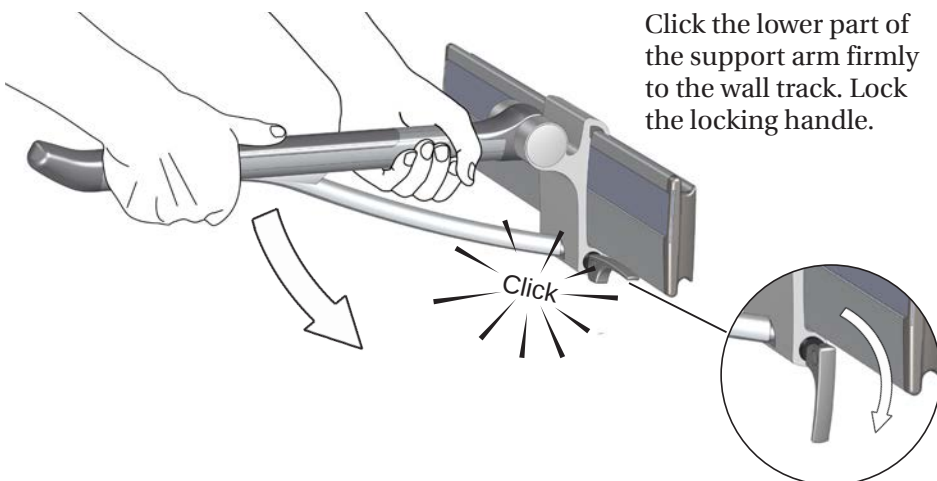Click the lower part of the support arm firmly to the wall track. Lock the locking handle.


Dismounting from vertical track

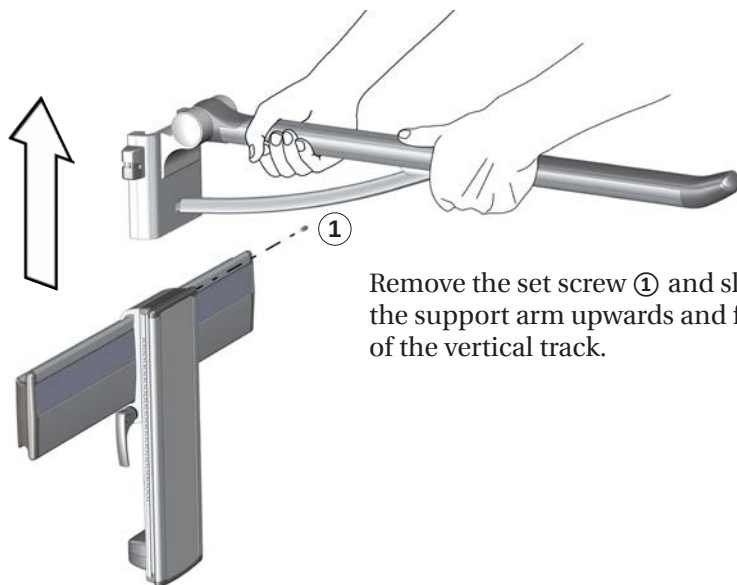Remove the set screw ① and slide the support arm upwards and free of the vertical track.

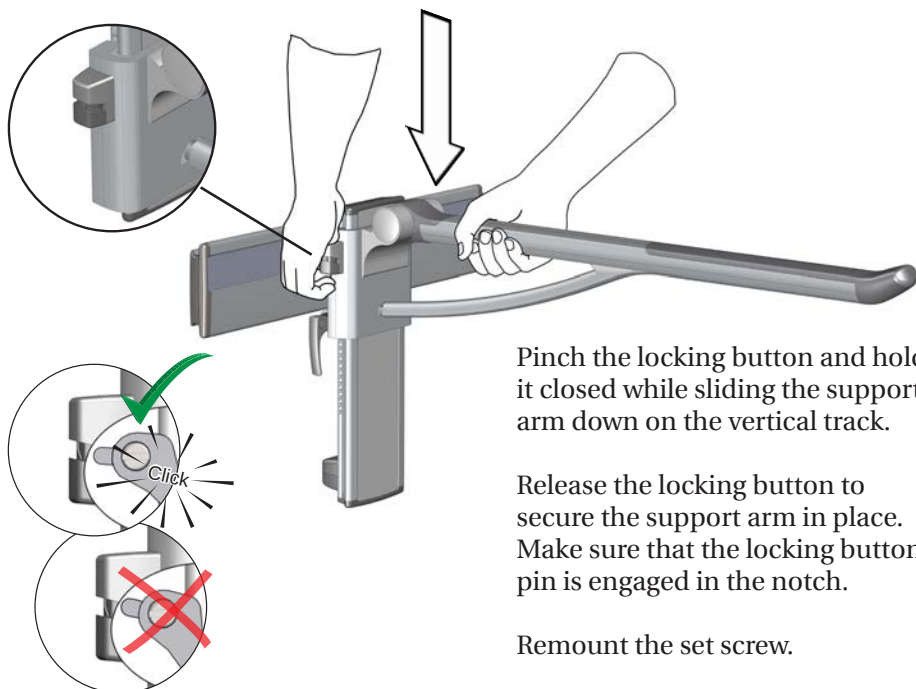Mounting on vertical track

Pinch the locking button and hold it closed while sliding the support arm down on the vertical track.

Release the locking button to secure the support arm in place. Make sure that the locking button pin is engaged in the notch.

Remount the set screw.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents tested by Pressalit can be found on www.pressalit.com.

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the product is in use.

Maintenance

Remove dust from the wall mount of the sideways adjustable support arm at a regular basis. Wipe with lubricant.
Expected service life of the support arm is 10 years.

Trouble-shooting


Symptom	Cause	Action
The product can't be moved sideways.	There might be dirt or dust on the horizontal walltrack.	Clean the walltrack and grease it with silicone.
The locking button doesn't lock.	The lock is not properly adjusted.	See mounting instruction.

Technical data


Max user weight	150 kg / 331 lbs
Materials	Anodized aluminium Stainless steel SEBS ABS


en

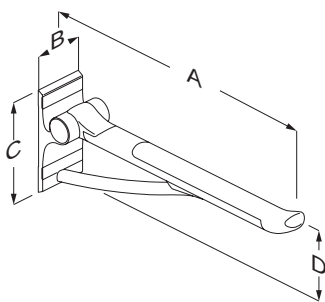Dimensions, mm


		
	R360070	R360085
A	700	850
B	100	
C	210	
D	168	

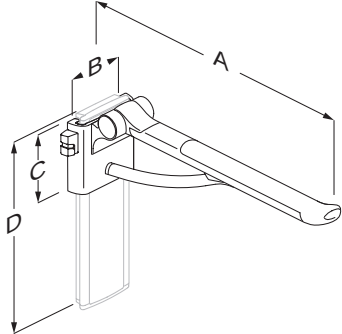en

				
	R371370 R371470	R371385 R371485	R370370 R370470	R370385 R370485
A	880	730	730	880
B	128			
C	150			
D	424			

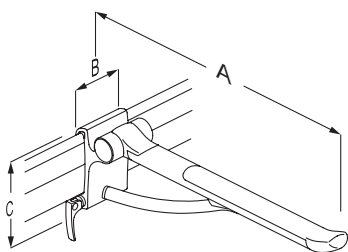	R381470	R381485
A	700	880
B	104	
C	175	

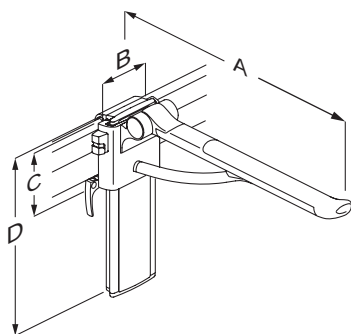	R391370	R391385	R391470	R391485
A	770	770	920	920
B	128			
C	150			
D	424			

en

Product weights

Model number	Product mass [kg]	Product mass (lbs)
R360070	2,78	6.12
R360085	3,10	6,83
R371370	2,78	6.12
R371385	3,34	7.36
R371470	2,78	6.12
R371485	5,20	11.46
R370370	4,00	8.81
R370385	4,34	9.57
R370470	4,00	8.81
R370485	4,34	9.57
R381470	3,10	6,83
R381485	3,10	6,83
R391370	4,01	8.84
R391385	5,20	11.46
R391470	4,01	8.84
R391485	3,34	7.36

en

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Aluminium	Aluminium is theoretically 100% recyclable without any loss of its natural qualities. It requires only 5% of the energy used to produce primary aluminium. The quality of recycled aluminium is so high that it can be used again and again, even in the case of anodised products as the anodised layer is a natural part of the aluminium.
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
SEBS	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy, which can be used for, for example, combined heat and power production.
ABS	As virgin ABS is somewhat expensive, recycling ABS is economically very attractive. Recycled ABS can be blended with virgin material to produce products with lower cost while preserving the high quality.

en